Rehab charger

Användaranvisning	(S)	2
User Manual	(GB)	3
Brugervejledning	(DK)	4
Bruksanvisning	(N)	5
Käyttöohje	(FI)	6
Charging Characteristic		7

Rehab laddaren

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Rehab Charger

User Manual

- Read this instruction before the charger is taken into use.
- Keep this manual within easy reach for the user of this battery charger
- Hydrogen gas will be produced when charging lead-acid batteries and hydrogen gas is explosive.
- Open flames and sparks should be kept away from batteries they may produce explosions.
- This charger is designed for cyclic applications without parallel load.

General

The Rehab Charger is suitable both for Freely Ventilated- (Wet) and Valve Regulated- (Dry) lead/acid batteries. The charger is small and very compact and can therefore easily be fitted close to the battery.

The charger operates in high frequency and should be connected to a standard 230V mains connections. The built in micro controller controls the charging process according to the charging algorithm.

During charging the charging progress is displayed with status indicator LED on the front panel. The micro-controller is also controlling the charging progress with regards to temperature in the charger and time.

If you have a special requirement of charging algorithm or application, please contact Exide Technologies – www.exide-nordic.com.

Installation

The Rehab Charger is suited for indoor and outdoor use.

It can be used positioned horizontally on a table or a shelf, hang on a wall or as a built-in charger in the vehicle or machinery.

The charger can be with fixed connection to the battery or with conventional connectors.

Position the charger in such a way that the air supply will not be obstructed.

When the charger is fitted on a wall or mounted in a vehicle it should be vertical mounted for optimal cooling.

The battery charger metal housing will exceed 41 degrees on the surface. If installed onboard a wheelchair it shall be installed out of reach or with a complementary cover.

Function

Check that the polarity between the charger and the battery is right. Positive + to positive + and negative – to negative -. The charging starts and the status indicator, Yellow LED, is lit up.

Yellow LED remains on until the battery is fully charged. If the charger is connected to a fully charged battery the charger will be charging for 1 hour. This is the minimum charging time. During this phase the green led will be illuminated. When only green LED is on the charger is kept in maintenance charger and ready to be used.

Maintenance charging will continue as long as the battery is connected.

The charger will be reset as soon as the charger is switched off and on, or if the battery is disconnected.

The charging time depends on the size of battery and the depth of discharge.

A freely ventilated battery (Wet) can be discharged to max. 80% and a valve regulated (Dry) with max 70%.

Trouble shooting and service

Yellow LED (flashing)	May mean that the battery is not properly connected. Check cables, connection terminals, plugs and other connections to the battery. Measure the voltage at the battery and at the charger.
Yellow and Green LED (simultaneous flashing)	The charging voltage is low <1,4V/cell
Yellow and green LED (alternate flashing)	Internal temperature is too high, > 60° C
Green LED (flashing)	High battery voltage or charger failure. If connections and battery voltage is ok, then contact Exide Technologies - www.exide-nordic.com
Red LED (flashing)	Battery defective

Charging algorithms

The Rehab charger is designed for freely ventilated and valve regulated batteries. If Exide Technologies is informed of battery capacity and type the charger will be delivered with the correct programming.

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Kun ilmoitat Exide Technologies Oy:lle akun tyyppin ja kapasiteetin, niin varaaja toimitetaan valmiiksi ohjelmoituna.